Nitwits®

all-in-one head lice treatment

Quick,
easy
IT WORKS!

IMPORTANT INTENDED PURPOSE: NITWITS ALL-IN-ONE IS A MEDICAL DEVICE DESIGNED TO KILL HEAD LICE AND THEIR EGGS
DIRECTIONS FOR USE: USE ON PATIENTS 6 MONTHS AND OLDER FOR THE TREATMENT OF HEAD LICE AND EGG INFESTATION

DIRECTIONS

READ ALL INSTRUCTIONS PRIOR TO STARTING APPLICATION

PREPARE

Cover shoulders with a towel and unlock spray nozzle by pressing button. Ensure hair is DRY.

STEP 1: SPRAY

Hold spray applicator 10cm from hair and spray with NitWits All-In-One until hair and scalp are evenly coated and saturated.

Massage well into scalp and through the hair to ensure complete coverage.

Leave treatment in hair for 20 minutes to ensure both lice and eggs are killed.

Rinse hair thoroughly and shampoo as normal.

STEP 2 (optional) : COMB

Use the comb provided to ensure all dead lice & eggs have been removed after treatment. This will ensure the hair is left especially clean.

AFTER 7 DAYS: Check the hair and scalp to ensure that there has been no new infestation. If lice are detected, repeat treatment.

CAUTION

Always read the label. Use Only as directed. Keep treatment solution out of reach of children. For external use only. Avoid contact with eyes. If contact does occur, flush with water. If symptoms persist see your healthcare professional. **Follow the directions for use, perform a test patch on a small area of the scalp. If after 24 hours there is no adverse reaction, proceed with a full application.** Do not use on broken or irritated skin. NitWits All-In-One is not designed to be used with any other device. Do not use on children under 6 months of age without first seeking medical advice. If irritation occurs discontinue use. Product may cause surfaces to become slippery. Take care and wash any residue away with water and detergent. Store below 30°C. **Do not use past the expiry date on the packaging.**

Any serious incident that occurs in relation to NitWits All-in-One should be reported to the manufacturer, and the competent authority of the Member State in which the user and/or patient is established.

Dispose of packaging thoughtfully and recycle where possible. Carton and IFU 100% recyclable. For further information on head lice, tips & treatments go to nitwitsforkids.com

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SI&D (Aust) Pty Ltd 29-31 Solent Circuit, Norwest, NSW, 2153 Australien, 1800 888 815
AU: Sponsor: N-Hance Pty Ltd, 29-31 Solent Circuit, Norwest, NSW, 2153
NZ: Vertrieben von: SI&D (NZ) Ltd, 24B Moorefield Rd, Johnsonville, Wellington, 6037, 0800110257