

NitWits®

Head Lice Comb

IMPORTANT NitWits Head Lice Comb is a Medical Device designed to detect and remove head lice and their eggs from the hair shaft

DIRECTIONS FOR USE: READ ALL INSTRUCTIONS PRIOR TO STARTING APPLICATION

WET AND DRY COMBING

1. Select long or short tooth comb for hair type.
2. Run the detangling teeth of the Head Lice Comb through the hair to smooth out knots and tangles.
3. Wet hair if using a conditioner, then divide hair into 4cm wide sections and comb through from root to tip with the metal comb. Wipe any lice or eggs off the comb and onto a tissue. *Tip: The short tooth comb is best to remove headlice eggs.*
4. Continue combing until the hair appears clear of eggs and lice. *Tip: Use the magnifying glass for a more thorough check.*
5. To ensure the headlice life cycle has been broken, steps 2 to 4 should be repeated every 3 to 4 days for 14 days to remove the lice.
6. Rinse hair thoroughly with warm water and shampoo as normal. Wash NitWits Head Lice Comb in warm soapy water and dry.

USING A TREATMENT SOLUTION

1. We recommend using the NitWits All-In-One Head Lice Solution. Follow the directions to Kill Head Lice and their Eggs.
2. Use Nitwits Head Lice Comb to remove the dead eggs from the hair shaft.

CAUTION

Always follow the directions for use. If symptoms persist seek advice from a healthcare professional. Do not boil the comb, Do not use if comb is broke. Store in a dry place. Unsuitable for use on children under 6 months of age.

Any serious incident that occurs in relation to NitWits Head Lice Comb should be reported to the manufacturer, and the competent authority of the Member State in which the user and/or patient is established.

Dispose of packaging thoughtfully and recycle where possible. Carton and IFU 100% recyclable and the plastic packaging can be recycled in systems that accept PE.



For further information on head lice, tips & treatments go to nitwitstorkids.com

Contact information

UK SI&D (Aust) Pty Ltd 29-31 Solent Circuit, Norwest, NSW, 2153 Australia, 1800 888 815

AU: Sponsor: N-Hance Pty Ltd, 29-31 Solent Circuit, Norwest, NSW, 2153

NZ: Distributed by: SI&D (NZ) Ltd, 24B Moorefield Rd, Johnsonville, Wellington, 6037, 0800110257

EC: REP Obelis s.a Bd General Wahis 53 1030 Brussels Belgium. Tel:+(32) 2 732-59-54 Fax:+(32)2 732-60-03 mail@obelis.net

IFU prepared on the 4th of October 2022.

